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ENG – Chlamydia and gonorrhoea test

- Perform the test at least 5 days after having unprotected sex.
 - Sampling is not recommended during heavy menstrual bleeding.
 - The test tube contains liquid that should not spill.
 - **Do not wash your genitals before taking the sample.**
1. Open the protective pouch at the shaft end of the sampling swab. Take hold of the shaft and pull the swab out of the pouch. Do not touch the absorbent sampling end of the swab with your fingers.
 2. Insert the swab about 5 cm into your vagina and rotate it so that the absorbent end rubs against the vaginal wall for approximately 30 seconds. Remove the swab from your vagina but do not put it down.
 3. Take out the sample tube and open it. Insert the absorbent end of the swab into the preservative solution in the sample tube. Bend the shaft to break it off at the score line. Leave the sampling end of the swab in the tube.
 4. To prevent the liquid from leaking out during transportation, screw in the cap of the sample tube firmly. Turn the tube up and down 3–4 times to mix the sample with the preservative.
 5. Put the tube into the resealable plastic bag. Do not remove the small piece of filter paper inside the bag. Seal the bag and put it into the box.
 6. Close the box carefully by inserting the wing-lets into the slots in the corners. Send the box in via regular mail. The postage is prepaid and no stamp is needed.

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CHECKING THE ANALYSIS RESULT

Once your sample has been analysed, you'll receive a text message. You can check your results by logging in to the page [See test results](#) with your bank ID.

Please note that if you ordered the test from a website for your municipality or FSHS, you will need to log in from that specific page to see your results.